Ovaj prijevod sastoji se od Stranica 8 / Listova 18

Broj-OV: 0381/2016

Datum: 27.06.2016.

Naziv dokumenta: Upute za održavanje Primus 1058 Life

Ime: VIK-Dental d.o.o.

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
U Zagrebu, 27.06.2016.

M. Nekich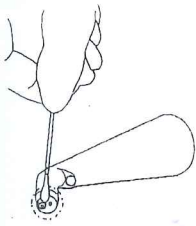

- Carefully wipe off contamination from the cannula tip with a soft Q-tip or soft cloth and water.


⚠ CAUTION

Damage caused by using a nozzle needle that is too long.

Damage to the internal hoses of the cannula.

- Use a short nozzle needle (Mat. no. 1.004,4986) to clean the media exit holes.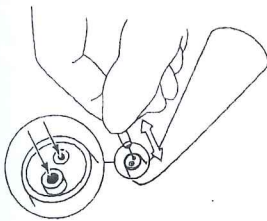

- Carefully remove contamination from the media exit ports using the short nozzle cleaning needle.

Disinfection of the 3-function and multifunction handpiece

Manual disinfection

External disinfection

- Wipe-disinfect the surfaces with a soft cloth.

Automatic disinfection

Not applicable.

⚠ CAUTION

Damage caused by improper cleaning/disinfection.

- Do not wash the gripping sleeve and the cannula in the thermidisinfector.
- Do not sterilise the gripping sleeve and the cannula in the hot-air steriliser.
- Do not place the gripping sleeve and the cannula in a disinfectant or ultrasonic bath.

Drying the 3-function and multifunction handpiece

Manual drying

- Blow off the outside and inside with compressed air until water drops are no longer visible.

Automatic drying

Not applicable.

Care of the 3-function and multifunctional handpiece

Requirement

The multifunctional handpiece has been reprocessed.

- ▶ Apply KaVo silicone grease (Mat. no. 1.000.6403) or KaVo Rota Spray 2 (Mat. no. 0.411.7510) to the O-rings of the joint between the gripping sleeve and cannula. Use cotton buds for this purpose.


Sterilising the 3-function and multifunction handpiece

Note

The grip sleeve and cannula can be sterilised.

- ▶ Hold the 3-way or multifunctional handpiece at the gripping sleeve and take off the cannula with a slight twisting motion.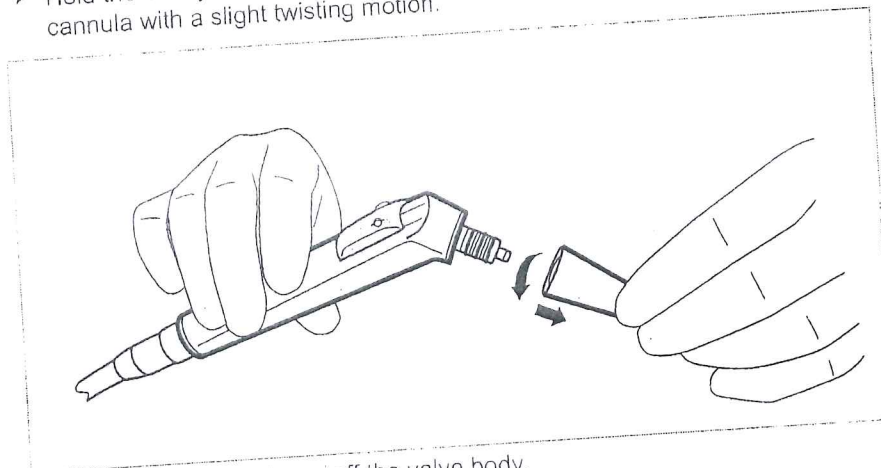

- ▶ Pull the gripping sleeve off the valve body.

Sterilization in a steam sterilizer in compliance with DIN EN 13060

- ▶ Weld the cannula and grip sleeve in sterile goods packaging.
- ▶ Sterilise with a triple fractionated vacuum (B class steriliser); hold for 4 minutes; $134^{\circ}\text{C} \pm 1^{\circ}\text{C}$; overpressure: 2.13 bar.

CAUTION

Damage caused by over-heating.

Damage to the gripping sleeve.

- ▶ Immediately remove the parts from the steriliser after the sterilisation cycle is completed.

Storage

- ▶ Reprocessed products should be stored protected from dust with minimum exposure to germs in a dry, dark and cool place.

2.4.5 Preparation of the instrument hoses

Cleaning and disinfection of instrument hoses

- ▶ Clean the hoses and couplings with a cloth and water.

CAUTION

Damage caused by spray disinfection.

- ▶ Do not subject instrument hoses to spray disinfection.
- ▶ Subject instrument hoses to wipe disinfection.

- FD 322 (Dürr Dental)
- Incidin Liquid (Ecolab)

Note

The exterior of the camera and camera hose can be disinfected with chemical disinfectants.

Comply with the manufacturer's instructions for use with regard to the application range, and exposure time.

**CAUTION**

Material damage due to use of non-approved disinfectant.

Damage to the camera.

- Use approved disinfectants only.

2.5 Reprocessing of the assistant unit**2.5.1 Preparing the instrument holder**


The instrument holder on the assistant unit is prepared in the same way as the instrument holder on the dentist unit.

See also:

- ▣ 2.4.2 Reprocessing of the instrument holder of the TM table, Page 11

2.5.2 Preparing the connector for the suction hose

- Remove connectors ① and ② from the suction hoses.

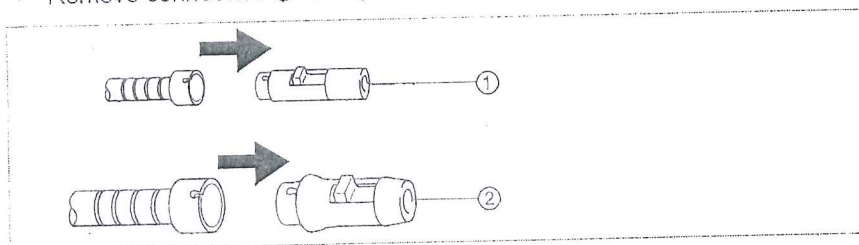**Cleaning and disinfection of the suction hose connector**

- Clean the disassembled parts under running water.
- Thermodisinfect the disassembled parts for at least 10 minutes at 93 °C (total cycle: approximately 20 minutes)

**Sterilising the suction hose connector****CAUTION**

Damage caused by sterilisation.

- Do not sterilise the suction hose connector.

**Note**

Sterilise the suction cannulas according to the manufacturer's instructions.


2.5.3 Reprocessing of sieve inserts and suction hoses

Note

Check the sieve inserts daily and after each surgery. The sieve inserts must be replaced at the latest when the suction decreases (Mat. no. 0.763.3423).
Cleaning is not recommended for hygienic reasons.

- ▶ Remove the suction hoses from the holder.
- ⇒ The suction motor starts automatically when the suction tubes are removed from the holder.
- ▶ Hold the cloth under the sieve seat, and pull the sieve insert out of the housing.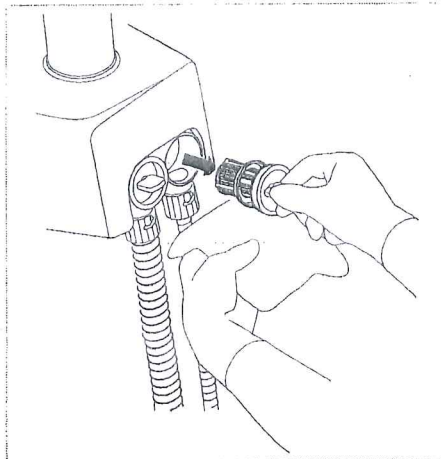

- ▶ If needed, insert a new sieve insert (Mat. no. 07633423) into the seat.
- ▶ Wipe the suction tubes with disinfectant.

2.6 Preparation of the patient unit

2.6.1 Preparation of the tumbler

- ▶ Reprocess as specified by the manufacturer.

2.6.2 Preparation of the tumbler holder and filler

⚠ CAUTION

Water in the device.

Material damage from leaking water upon actuation of the "Tumbler filler" or "Bowl flush" keys.

- ▶ Make sure that the unit is turned off before removing the tumbler filler or the spittoon bowl.
- ▶ Always check the correct seating of tumbler filler and spittoon bowl before turning on the unit.

Cleaning and disinfection of the tumbler holder and filler


Note

When assembling, make sure that the O-ring is present on the tumbler filler.

Note

Inspect the O-ring (Mat. no. 0.200.6128) for damage and exchange if necessary to prevent water or oxidation damage.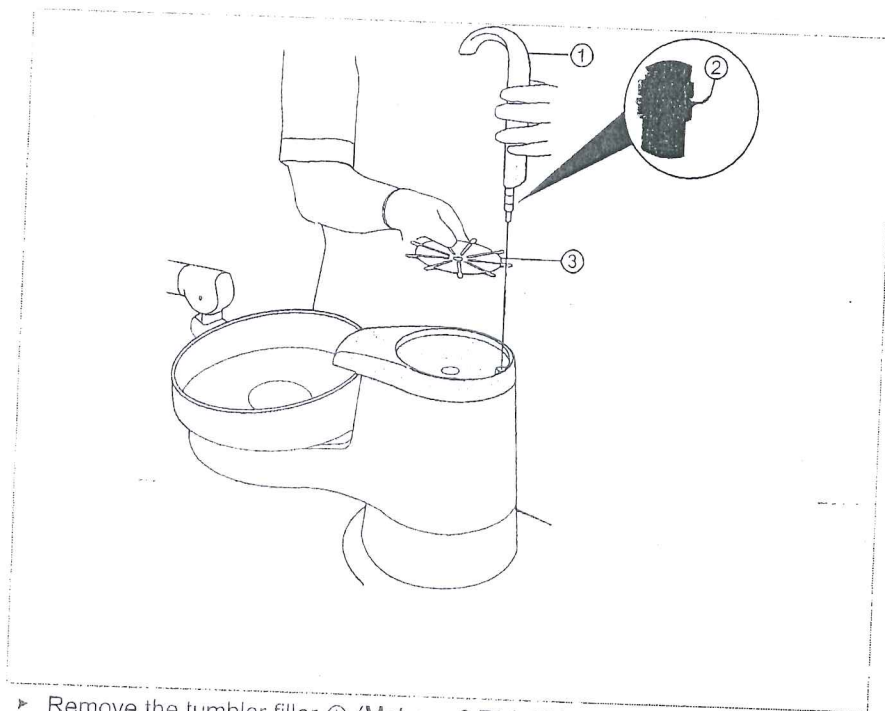

- ▶ Remove the tumbler filler ① (Mat. no. 0.764.8541) by pulling upwards.
- ▶ Take off the tumbler holder ③ (Mat. no. 07646153).
- ▶ Clean the tumbler holder and tumbler filler under running water.
- ▶ Wet the O-ring ② (Mat. no. 02006128) with silicone grease (Mat. no. 10006403).

CAUTION

Damage caused by thermidisinfection.

- ▶ Do not subject the tumbler holder to thermidisinfection.

The tumbler filler can be subjected to thermidisinfection.

Sterilisation of the tumbler holder and filler

CAUTION

Damage caused by sterilisation.

- ▶ Do not sterilise the tumbler holder and filler.

2.6.3 Preparation of the spittoon

CAUTION

Water in the device.

Material damage from leaking water upon actuation of the "Tumbler filler" or "Bowl flush" keys.

- ▶ Make sure that the unit is turned off before removing the tumbler filler or the spittoon bowl.
- ▶ Always check the correct seating of tumbler filler and spittoon bowl before turning on the unit.


Cleaning and disinfection of the spittoon bowl

CAUTION

Damage caused by inappropriate cleaning agents.

- ▶ Do not use acid-containing, sand-containing, and form-forming cleaning agents.
- ▶ Add a dose of DEKASEPTOL gel to the spittoon bowl and distribute the agent with a brush.
- ▶ For cleaning, allow the gel to act for approx. 5 minutes.
- ▶ For disinfection, allow the gel to act for approx. 15 minutes.
- ▶ After letting it work, press the "Spittoon bowl" button.
- ▶ Clean the spittoon bowl thoroughly using a brush.
- ▶ For thorough cleaning or thermidisinfection, rotate the spittoon bowl and lift it off.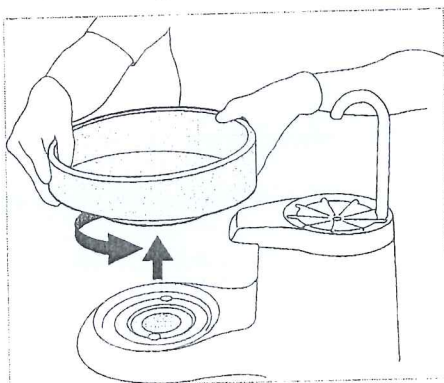


CAUTION

Damage caused by thermidisinfection.

- ▶ Do not subject spittoon bowls made of glass to thermidisinfection.

The spittoon bowl made of porcelain can be subjected to thermidisinfection.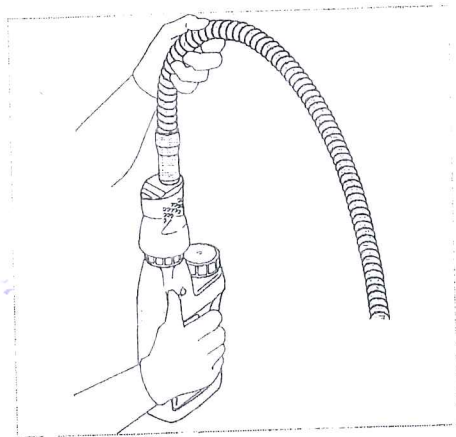

2.9.2 HYDROclean function

The HYDROclean function is an automatic programme for hydro-mechanical cleaning of the drainage and separation systems within and outside the treatment unit. The cleaning process is carried out by the supply of fresh, clear water (limited rinsing). The water is aspirated from the Aquamat through the filter housing to the separation system. The spittoon bowl rinsing is alternately turned on and off. The HYDROclean function prevents the coagulation and adhesion of proteins inside the drainage system and improves the transporting of fine particles.

Carrying out the HYDROclean function:

- After each treatment

Preparation

Requirement

Suction hoses remain hanging in the holder.

- ▶ Press the "HYDROclean" button.

⇒ The HYDROclean function is in the preparatory stage.

Start the HYDROclean function

- ▶ Press the "HYDROclean" button.

⇒ This starts-up the HYDROclean function.

The HYDROclean function is completed when the top LED goes dark.

The process can be discontinued at any time by pressing the "HYDROclean" button.


2.10 Preparation of the amalgam separator

The following systems can be used as options:

- "Dürr" Amalgam separator
- Preparation for external wet suction for connection to a central amalgam separator
- "Dürr" separation kit

Removing and cleaning the water bottle

- ▶ Slowly turn the water bottle counterclockwise to take it off.

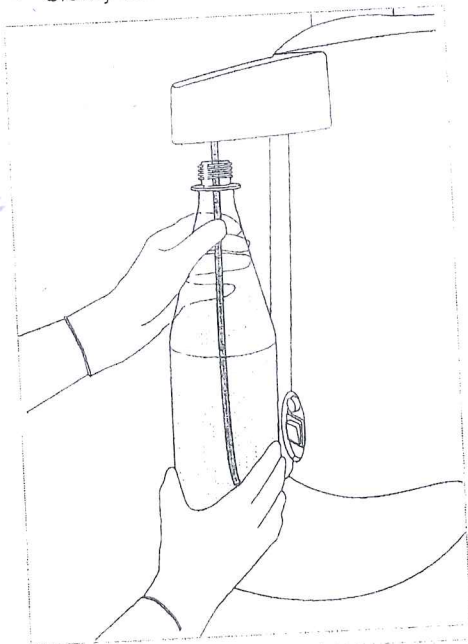Note

This allows the over-pressure to be released slowly from the water bottle.

Note

The water in the water bottle needs to be changed at least one daily. In order to ensure the efficiency of the measures, KaVo urgently recommends reprocessing the water with KaVo OXYGENAL 6.

- ▶ For cleaning, rinse the water bottle with water at less than 50 °C.

Filling the water bottle with the dosing facility.

- ▶ Wipe up any spilled OXYGENAL immediately in order to prevent discolouration of the surfaces.
 - ▶ Remove the protective hose from the tube of the KaVo OXYGENAL 6 dosing facility.
 - ▶ Guide the tube through the orifice of the coarse sieve of the KaVo OXYGENAL 6 bottle.
 - ▶ Screw the KaVo OXYGENAL 6 dosing facility tightly to the KaVo OXYGENAL 6 bottle.
 - ▶ Rotate the orifice in the lid to a filling level of 1.5 litres.
- ⇒ Lid snaps-in in the filling position.