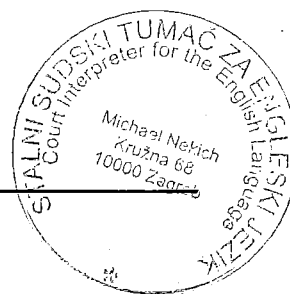
Ovaj prijevod sastoji se od Stranica 4 / Listova 10
Broj-OV: 0774/2016
Datum: 11.10.2016.
Naziv dokumenta: E10S8 Life TM/C/S
Ime: VIK-Dental d.o.o.

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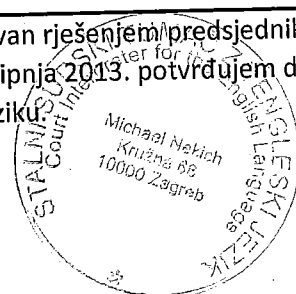
7.7 Device description

Dentist element

E1058 Life TM/C/S. Left-handed or right-handed option can be determined at the factory.

1058 S/TM. Instrument carrier with easily moving spring or pivot arm with wide swivelling range, adjustable in height, with 5 instrument holders, locking brake and integrated pneumatic spring.

1058 Life C. Height-adjustable Cart on wheels, with 5 instrument holders

Electrical instrument holder

Instrument delivery system equippable with 2nd to 5th pneumatic outlets, KL 701 motors or KL 703 LED motors (brushless DC motor), ultrasonic scaler, intraoral camera

Control element with LCD display on the instrument table for instruments and patient chair functions

Clock and timer display

Automatic shutoff of the unit-mounted operating light when the patient exits up from the chair or is placed in the rinsing position as well as in programme positions 1 and programme position 2

Dentist table handle on S, removable, thermoisinfectable

Handpiece holders, removable for cleaning

Smooth, wipe-down instrument tubing with threaded, quick-release couplings - easy to mount

Exhaust air filter for instrument return air

Integrated anti-suck back water system, with automatic air-purging

Additional equipment for the dentist element

MEMOSpeed - indication of speed or intensity storage for KL motors and PIEZOsoft/PIezoLED for 2 users as well as clock and timer

Triple function handpiece or Multi functional handpiece with light

Tray holder (for 2 norm tray, 1 norm tray ord US tray)

Non-slip mast for tray holder, removable, thermoisinfectable

2nd handle (S only)

Holder for waste receiver

Spray heating for instruments

PiezoLED ultrasonic scaler / PIEZOsoft ultrasonic scaler

Intraoral camera ERGOcam One, 2. to 5. position

X-ray viewer for image size 5x5 cm

Assembly kit multi functional handpiece, and intraoral camera standing

Assistant element

Comfort assistant's element for height-adjustable arm pivotable, with holder for triple function handpiece, curing light, 1x saliva ejector tubing, 1x HV suction tubing incl. electric holder switch and membrane control element, identical to instrument table for instrument, spittoon and chair functions

Spittoon on unit body, with pivotable spittoon support and safety cut-off device in the bowl support

Electrical control for tumbler wash and bowl flush functions

Porcelain bowl, pivotable towards the patient. Porcelain bowl, removable, thermoisinfectable

Compact filter housing, with connection for suction tubes and easily exchangeable suction filter

Suction tubing set with HV suction tubing, saliva ejector tubing and suction cannula set

Central supply element, integrated in unit body, flush or surface mounted

7 KaVo Primus™ 1058 Life TM/S/C

KaVo drinking water component and germ reduction system: see Optional Extras
 Compressed air filter with automatic condensed water separator and pressure spring
 Central electronic power supply with overload protection, for unit and chair control
 Mains voltage conversion for 100/110/120/220/240 V - 50 Hz/60 Hz

Additional equipment for the assistant element

Assistant's element, pivots R/L with holder for triple function handpiece, curing light Satelec MiniLED,
 1x saliva ejector tubing, 1x HV suction tubing
 Additional tray holder for one Norm tray or kit for DIAGNOdent holder
 Triple function handpiece for assistant
 Multi function handpiece for assistant
 Curing light can be connected to assistant's element
 Kit for 2nd saliva ejector
 Selective support kit
 Kit for suction tubing guidance roller

Patient chair

Height adjustment by spindle motor drive, min. 350 mm - max. 830 mm stroke
 Maximum patient weight loading up to 185kg
 Backrest/seat tilt by spindle motor drive, 1° - 70° adjustment range
 Trendelenburg movement (seat movement synchronously to backrest)
 Offset articulation point for avoiding the "stretching effect"
 4 programmable AP positions (Automatic Position)
 LP position (last position) and SP position (rinsing position)
 Safety shut-off in the seat, backrest, step plate
 Upholstery set in soft, elastic design, changeable, colours according to KaVo colour chart
 Backrest in "Progress" design, with reduced width in shoulder support region for optimal ergonomic procedure
 Double-jointed headrest, extendable, with mechanical tilt adjustment, ergonomic rotary control knob

Additional equipment for the patient chair

Armrest on left and/or right, can be swivelled up to ease entry and exit
 Double-jointed headrest press button with smooth, individual mechanical angle adjustment, even for more extreme patient head positioning; rotating cushion for pediatric treatment
 Additional comfort cushion headrest, turning cushion for upper / lower operating, suitable for long-term treatment, with heat-sensitive cushioning material optimum match measurement of the pad to the head shape
 Patient chair COMPACT chair with bendable foot bench instead of patient chair Standard. Backrest/seat inclination with spindle motor drive, 1° - 85° adjustment range

Unit body

Multi-functional foot control, electrical, for smooth speed regulation with preselected spray and chip blower function. When equipped with an electric motor, switchover to counterclockwise motor operation, possible. Full hygienic control of patient chair when instruments de-selected

Safety shut-off in the foot control stirrup, wireless foot control
VACUstop for briefly turning off the suction during retraction by actuating the patient chair base switch
Porcelain bowl/glass bowl, removable, can be swung towards patient
Connection to various suction systems (one system must be present). Position selection valve is included in the kits - except for solids collector kit
Compact filter housing, with connection for aspiration tubings and easily exchangeable suction filter
KaVo tap water component with free drain, DVGW or EN1717 tested - test ref. number DVGW NW-0402BT0111 - with integrated water germ reduction system with electronic monitoring of the germ reduction container/acoustic refilling signal
Alternatively, compact water block without DVGW*, with water filter and shut-off valve
Alternatively, water bottle with compact water block with DVGW (for independent water supply from the water mains)
Possibility for mounting upright post for an operating light on the unit body. The unit can also be ordered for left-handed users
* DVGW (= German Association for gas and water management)

Additional equipment unit body

Monitor holder to hold monitors on the unit-mounted light mounting post
24 V warm water heater for tumbler filler
Steel mounting plate for right or left handed installation (for screwing-down the unit if floor conditions are difficult)
Kit for connecting of external devices with connections for water, air and electricity

7.8 Surgery layout

Important technical data

Mains voltage:	230 V/50/60 Hz, switchable to: 100/110/120/130/210/220/240 V
Control voltage:	24 V
Power consumption:	200-1500 VA
Water pressure:	2 bar min., 6.0 bar max.
Air pressure:	5.2 bar min., 7.0 bar max.
Water quality	Tap water
pH	7.2 - 7.8

Note:

Starting at a water hardness of 12° dH, we recommend installing a water treatment system. The local water authorities can advise you on the water hardness in their water supplies. Free drain into KaVo water components – test ref. number: DVGW DW-0402AT2146