Ovaj prijevod sastoji se od  
str. 13/listova 13

Datum: 01. 11. 2013

Broj ovjere: 362/2013

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Daniela Vujević, dipl.iur.


Ovlašteni sudski tumač za engleski jezik

A handwritten signature in black ink, which appears to read "D. Vujević".

to SONICflex® quick 2008/L • SONICflex® 2003/L

the efficient  
all-rounder  
for your practice.

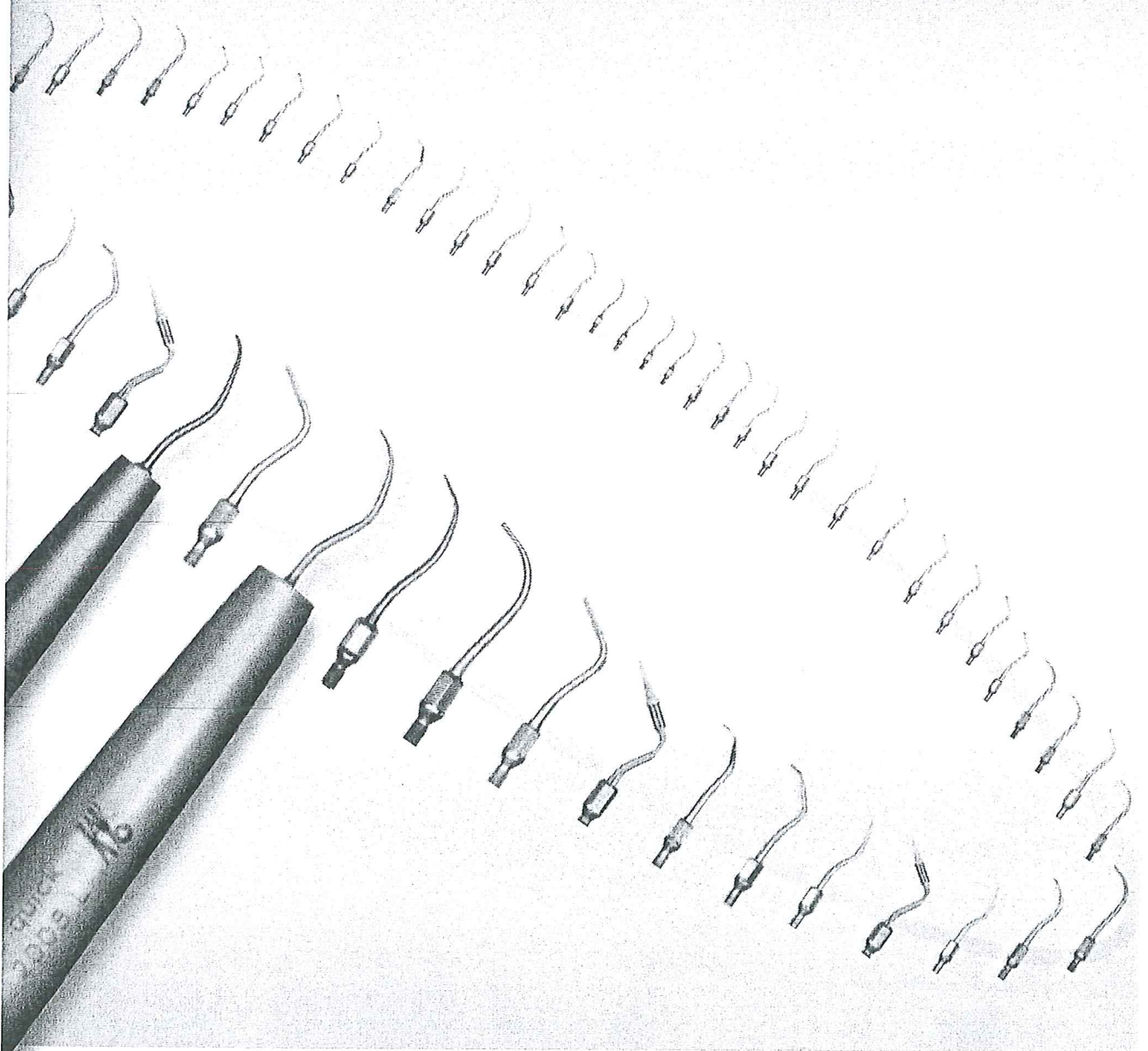## High versatility – developed to a fine point.


More and more patients understand the necessity of regular prophylaxis, for maintenance of healthy teeth and gums. Here, they particularly prefer gentle treatment. KaVo SONICflex is easy-to-use and permits comfortable treatment – the perfect handpiece for prophylaxis. This may even increase patients' loyalty to the practice and you will benefit from the long-term trend of prevention.

### The right fit for every use.

The wide range of options offered by the SONICflex will soon make it one of your most-used instruments. Choose the right tip for your needs from the wide range of types.


### High versatility – main features and advantages for you:

- More than 50 tips and more indications than any other air-scaler
- For every application
- Ensuring successful minimally invasive treatment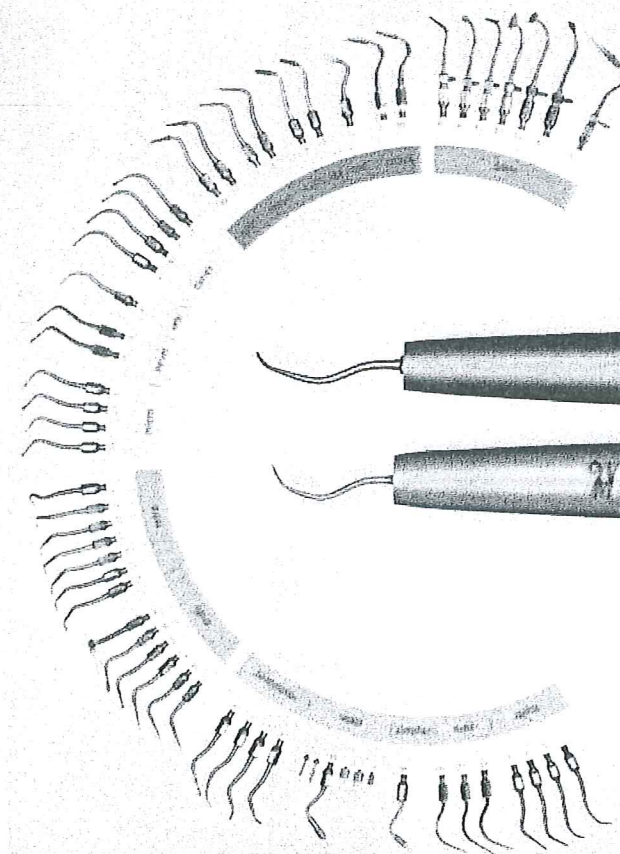

KaVo *Motion* Series

KaVo *Express* Series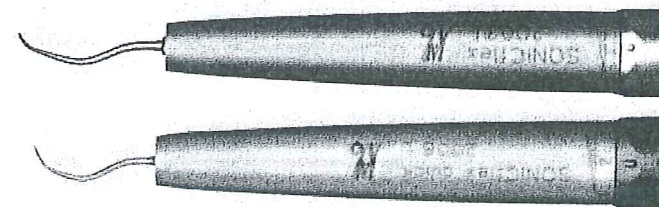



## Gentle treatment – comfortable for your patients.

### Patients who are treated comfortably enjoy coming back.

The KaVo SONICflex offers your patients much gentler treatment. The low noise and wide elliptical strokes, minimise the development of heat. In addition, tooth tissue is protected, as the handpiece operates with a low application pressure on the tooth. At all performance levels, treatment with the KaVo SONICflex is noticeably more pleasant and conserves healthy tooth tissue.

### Patient's comfort – outstanding features and advantages:

- Pleasant treatment
- Preserves tooth tissue
- Gentle oscillations, particularly suitable for pain-sensitive patients
- Low heat generation

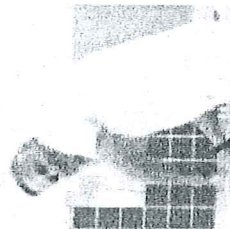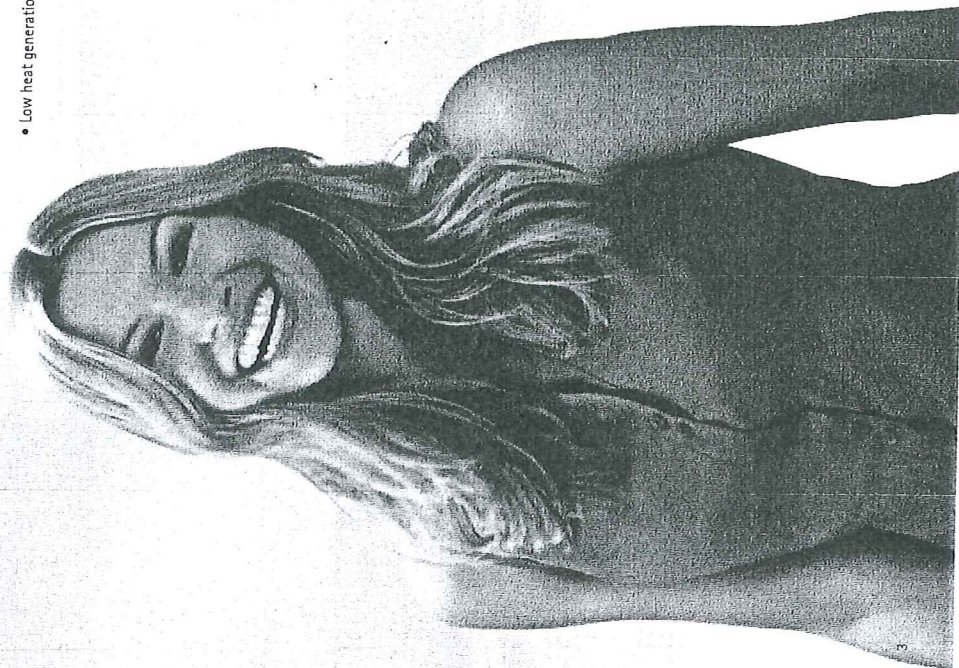
KaVo. Dental Excellence.

# Efficiency and high performance – just simple.

## Simply twist, for efficient working.

Tip-change is very simple with your KaVo SONICflex: thanks to a patented KaVo design, the tips can be easily and rapidly changed, with just one twist of the wrist. This saves valuable time and allows effective workflow, without unnecessary interruptions.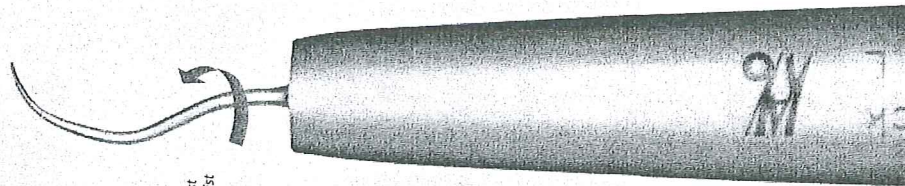

*KaVo-patented: easy and fast tip-change with just one twist of the wrist.*

## KaVo SONICflex® – main features and advantages for you:

- The most comprehensive spectrum of clinical indications, including gentle prophylactic treatment, of all sonically-powered scalers available on the market.
- Easy and fast tip-change – effective, saves time, and prevents unnecessary interruptions of your workflow
- Very high clinical power – up to 50% more than the predecessor model – difficult plaque is easily removed
- No additional equipment is necessary: just connect it to the KaVo MULTiflex or Sirona coupling, without any annoying, over-floor cables, or additional foot controls
- Optimum illumination: 25,000 LUX for clear visualisation, without eye-fatigue – also available with LED
- Substantially quieter than comparable models offered by competitors (up to 15 dB) – which translates into less stress for the dental team and patients
- Proven KaVo quality – the KaVo SONICflex 2003 L and 2008 L represent the third scaler generation from KaVo, the culmination of 25 years of knowledge and experience

## You won't fail to be impressed.

Your KaVo SONICflex is the most powerful and quietest air scaler of its class. The instrument with the most comprehensive range of uses. It supports your efforts with adjustable performance levels for a myriad of indications – strong and precise.


## KaVo SONICflex® quick 2008 L and KaVo SONICflex® LUX 2003 L – adjustable performance for any indication.

| Output levels | Amplitude | Frequency | Volume    | Indications                                   |
|---------------|-----------|-----------|-----------|-----------------------------------------------|
| Level 1       | 120 µm    | 6000 Hz   | 61 dB (A) | Fine finishing and particularly delicate work |
| Level 2       | 160 µm    | 6000 Hz   | 69 dB (A) | Preparation and plaque removal                |
| Level 3       | 240 µm    | 6000 Hz   | 71 dB (A) | Aggressive preparation and plaque removal     |

## Clearly reliable, ergonomic and hygienic.


Your KaVo SONICflex comes with an especially robust and easy to grip Plasmatec coating. This metal ceramic surface allows you to safely and securely position your KaVo SONICflex in your hand. Your fingers are more relaxed during treatment, and the grip is pleasantly warm.

The Plasmatec surface is provided with the KaVo hygienic seal. It may be processed in a Thermofinfecter and sterilised up to 135 °C. It therefore satisfies current guidelines for hygiene (such as RKI) – an important plus for the hygiene of your practice.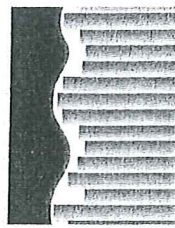

Hygienic seal  
Plasmatec coating  
Instrument sleeve

The Plasmatec surface coating technology used by KaVo provides both an excellent grip and outstanding hygiene.

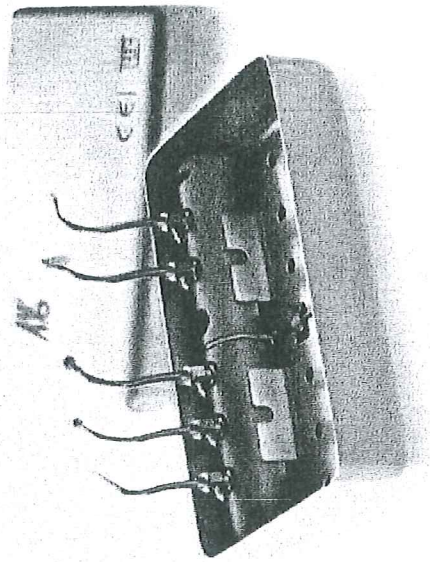KaVo SONICflex bone tip-set, with functional storage box.

KaVo. Dental Excellence.

# ntology and endodontics to ...

offer you support every day.

options are offered in your practice. The  
the KaVo SONICflex system, allows you to  
handpiece solution. Choose your personal  
and it as necessary. This allows for perfect  
situations - so easy and gentle for your

Perfect prophylaxis.  
So easy for you and  
so gentle for your  
patients.

## id periodontology

### SONICflex® scaler

- A variety of tips for tenacious calculus removal
- Clean shallow pockets

60

61

62

63

### SONICflex® perio

- Different tips for minimally-invasive, gentle and efficient cleaning of periodontal pockets up to 9 mm deep

### SONICflex® clean

- Wide spectrum vibrating brush
- For gentle cleaning and polishing

### SONICflex® perio accessories

- Coupling for the supply of therapeutic fluids such as sterile saline solution (Order No.: 1.000.6296)

## Prophylaxis and periodontology

48


### SONICflex® implant

- Effective removal of sub-gingival plaque and calculus
- No abrasion, no roughening
- Disposable cleaning attachment, to prevent the transmission of diseases


### SONICflex® root planer

- For easily processing of difficult-to-access or narrow furcations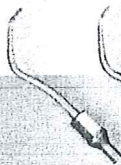


24

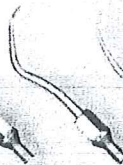25

26

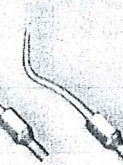27

## Endodontology

66

67

68

69

70

96

### SONICflex® endo

SONICflex endo - a special line of tips for:

- Probing for canals
- Removing soft materials during revision
- Fashioning canal inlets
- Shaping the pulp chamber for coronal root preparation
- Cleaning and rinsing of root canal (tip 96 available in ISO sizes 0.15, 0.20 and 0.25)

56

16

### SONICflex® retro

- Retrograde root preparation with contra-angle, diamond-coated tips
- Prevention of root damage from hairline cracks, support of tooth stability, protection of the periapical bone
- Preparation along a precise axis

20

55

57

17

21


KaVo. Dental Excellence.

# ... minimally invasive caries therapy and preparation ...

Everything. Easy. Gentle.

## Minimally invasive caries therapy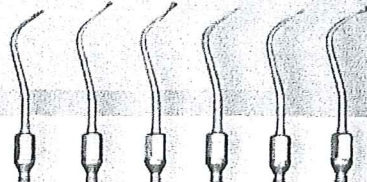


### SONICflex® microinvasive

- Special tip system for small, difficult-to-access lesions in the proximal region
- Tips diamond-coated on one side allow direct access to initial lesions from an occlusal or buccal direction
- Maintenance of healthy tooth substance


45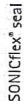


### SONICflex® seal

- Preparation of fissures
- The perfect complement to the KaVo DIAGNOdent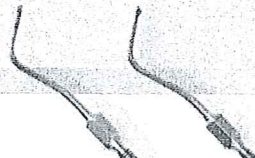

### SONICflex® bevel


- Tips diamond-coated on one side for the precise bevelling of prepared small lesions
- Optimisation of adhesion
- Minimisation of the marginal gap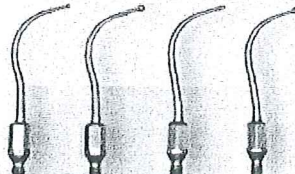

### SONICflex® caries D + TC


- Minimally-invasive removal of carious dentine
- Caries TC: Removal of large carious lesions
- Caries D: for small carious lesions with the ability to shape the cavity

## Preparation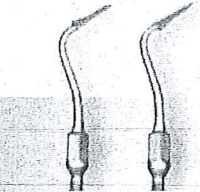


### SONICflex® prep gold

- Preparation tips diamondcoated on one side for finishing the edges of cavities, specially developed for gold inlays and partial crowns
- The attachment geometry for gold inlays and partial crowns is precisely transferred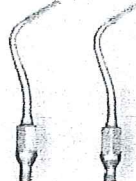


### SONICflex® prep ceramic

- For ceramic inlays/onlays
- For finishing the margins of cavities; directly transfers the geometry of the tips to the tooth substance
- No more undercuts with laboratory-fabricated inlays and onlays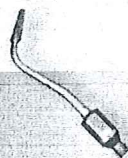

34

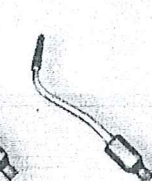35

### SONICflex® prep CAD/CAM

- Ideal shaping of the cavity margins for CAD/CAM systems such as EVEREST and CEREC

97

98

### SONICflex® prep crown

- For perfectly fine-finished crown preparations

12

### SONICflex® cem

- Gentle and precise inscription of inlays
- Reduction of the marginal gap

KaVo. Dental Excellence.

90/218

... to surgical applications.

## Surgery

80

81

### SONICflex® bone

- Can be used in many areas of bone treatment (osteotomies, augmentation, extractions and sinus lift)

82

83

84

85

### SONICflex® bone tip 85 (membrane disc)

- This disc-shaped form makes it ideal for reliably and gently repositioning the Schneider membrane during a sinus lift

86

### SONICflex® bone tip 86 (bone harvester)

- Ideal for easily and quickly harvesting a greater amount of bone, especially useful during a sinus lift procedure

87

### SONICflex® bone tip 87 (saw)

- Ideal for precise, extremely narrow incisions, when cutting ligaments etc.

# KaVo supplies the perfect solution for each and every step during prophylaxis:

So easy for you.

So gentle for your patients.

## 1. Workflow Integration (e.g. with KaVo Primus® 1058 TM)

- Professional prophylaxis system and reliable treatment centre in one
- Highest reliability for first-rate prophylaxis and treatment results
- Bracket table, swing-arm and cast version, with flexible equipment versions for all requirements

- Comfortable patient positioning with soft upholstery
- Create a feel-good ambience for your patients with attractive designs
- Effective patient marketing with optional multimedia solution

## 2. Scaling (e.g. with KaVo SONICflex quick 2008 L)



- Easy and fast tip change with just one turn
- One handpiece for an extremely large number of indications
- Also compatible with Sirona couplings

- Soft oscillating action, especially suitable for sensitive patients
- High patient confidence level due to gentle treatment
- Low noise air-scaler with light

## 3. Tooth cleaning (e.g. with KaVo PROPHYflex 3 - 2018)

- 360° rotating tip for ideal work conditions without tiring
- Entirely sterilisable and therefore easy to integrate into the hygiene workflow
- Also compatible with Sirona couplings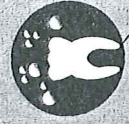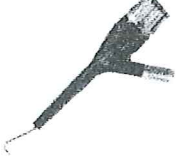

- Perfect tooth cleaning for visibly better results
- Gentle and pleasant treatment with PROPHYpearls
- 4 individual tastes (berry, cherry, mint and orange)

## 4. Polishing (e.g. with KaVo DURAflex 2033)

- Adaptable for both snap-on or screw-in instruments
- Special scaling, perfect for prophylaxis
- Best polishing results due to ideal paste distribution with 90° rotation

- Soft polishing with mild vibrations
- Noticeably gentle
- Cleaner teeth you can both see and feel

## 5. Caries detection (e.g. with KaVo DIAGNOdent)

- Perfect findings in every field of application
- Reliable detection of occlusal and proximal caries, plus calculus detection in perio-pockets
- Early detection of hidden caries via Quantitative LASER Fluorescence technology
- Wireless

- Preservation and protection of healthy tooth tissue
- Pain-free detection
- Higher patient confidence levels due to reliable diagnosis

# Comprehensive selection for your entire range of uses.

for SONICflex  
2000 N, 2000 L, 2003 / L

for SONICflex  
2008 / L, 2008 S / LS

| Application                           | KaVo tip name                                                                | No. | Mat. Number | No.   | Mat. Number |
|---------------------------------------|------------------------------------------------------------------------------|-----|-------------|-------|-------------|
| Calculus removal                      | SONICflex scaler tip, universal                                              | 5   | 0.571.5171  | 5A    | 1.005.8949  |
|                                       | SONICflex scaler tip, crescent                                               | 6   | 0.571.5181  | 6A    | 1.005.8950  |
|                                       | SONICflex scaler tip, perio                                                  | 7   | 0.571.5179  | 7A    | 1.005.8951  |
|                                       | SONICflex scaler tip, perio, extra-long                                      | 8   | 0.571.5371  | 8A    | 1.005.1953  |
|                                       | SONICflex scaler tip set                                                     | Set | 1.000.4983  | Set A | 1.005.9328  |
| Cleaning the root surface             | SONICflex tip for brush attachments                                          | 48  | 0.571.0401  | 48 A  | 1.006.1982  |
|                                       | SONICflex perio tip, long, straight                                          | 60  | 0.571.7402  | 60A   | 1.006.1934  |
|                                       | SONICflex perio tip, long, left                                              | 61  | 0.571.7412  | 61A   | 1.006.1935  |
|                                       | SONICflex perio tip, long, right                                             | 62  | 0.571.7422  | 62A   | 1.006.1936  |
|                                       | SONICflex perio tip set                                                      | Set | 0.571.0371  | Set A | 1.006.2020  |
|                                       | SONICflex root planer tip, bud, small, left                                  | 24  | 0.571.5621  | 24A   | 1.006.1957  |
|                                       | SONICflex root planer tip, bud, small, right                                 | 25  | 0.571.5631  | 25A   | 1.006.1959  |
|                                       | SONICflex root planer tip, button, small, universal                          | 26  | 0.571.5641  | 26A   | 1.006.1961  |
|                                       | SONICflex root planer tip, button, large, perio                              | 27  | 0.571.5651  | 27A   | 1.006.1963  |
|                                       | SONICflex root planer tip set                                                | Set | 0.571.0301  | Set A | 1.006.2022  |
| Cleaning the implant neck             | SONICflex tip implant                                                        | 48  | 1.003.8167  | 48A   | 1.006.2027  |
| Retrograde root canal treatment       | SONICflex retro tip, isthmus, left                                           | 56  | 0.571.7322  | 56A   | 1.006.2040  |
|                                       | SONICflex retro tip, isthmus, right                                          | 57  | 0.571.7332  | 57A   | 1.006.2041  |
|                                       | SONICflex retro tip cylinder, small, left                                    | 16  | 0.571.5541  | 16A   | 1.006.2036  |
|                                       | SONICflex retro tip cylinder, small, right                                   | 17  | 0.571.5581  | 17A   | 1.006.2035  |
|                                       | SONICflex retro tip T-shape, small, left                                     | 20  | 0.571.5521  | 20A   | 1.006.2037  |
|                                       | SONICflex retro tip T-shape, small, right                                    | 21  | 0.571.5561  | 21A   | 1.006.2038  |
|                                       | SONICflex retro tip, anterior tooth                                          | 55  | 0.571.7342  | 55A   | 1.006.2039  |
|                                       | SONICflex retro tip set                                                      | Set | 0.571.0381  | Set A | 1.006.2034  |
| Locating and preparing the root canal | SONICflex endo tip, button, small, diamond coated                            | 66  | 1.000.5825  | 66A   | 1.006.1992  |
|                                       | SONICflex endo tip, conical, 125°, diamond coated                            | 67  | 1.000.5822  | 67A   | 1.006.1994  |
|                                       | SONICflex endo tip, conical, 112°, diamond coated                            | 68  | 1.000.5823  | 68A   | 1.006.1996  |
|                                       | SONICflex endo tip, button, large, diamond coated                            | 69  | 1.000.5827  | 69A   | 1.006.1998  |
|                                       | SONICflex endo tip, conical, 117°, tip not diamond coated                    | 70  | 1.000.5821  | 70A   | 1.006.2000  |
|                                       | SONICflex endo tip set                                                       | Set | 1.000.5828  | Set A | 1.006.2031  |
|                                       | SONICflex endo clean tip 9G                                                  | 96  | 1.008.5163  | 96A   | 1.008.5164  |
|                                       | SONICflex endo clean set                                                     | Set | 1.006.2045  | Set A | 1.007.1142  |
|                                       | SONICflex endo clean needles 015                                             | -   | 1.006.2042  | -     | 1.006.2042  |
|                                       | SONICflex endo clean needles 020                                             | -   | 1.006.2043  | -     | 1.006.2043  |
| Minimally-invasive caries therapy     | SONICflex seal tip                                                           | 45  | 1.000.8323  | 45A   | 1.007.1506  |
|                                       | SONICflex micro torpedo tip, mesial, diamond coated                          | 28  | 0.571.6741  | 28A   | 1.006.1985  |
|                                       | SONICflex micro torpedo tip, mesial, diamond coated                          | 29  | 0.571.6731  | 29A   | 1.006.1987  |
|                                       | SONICflex micro tip, small hemisphere, mesial, diamond coated                | 30  | 0.571.6811  | 30A   | 1.006.1969  |
|                                       | SONICflex micro tip, small hemisphere, distal, diamond coated                | 31  | 0.571.6801  | 31A   | 1.006.1971  |
|                                       | SONICflex micro tip, large hemisphere, mesial, diamond coated                | 32  | 0.571.6831  | 32A   | 1.006.1973  |
|                                       | SONICflex micro tip, large hemisphere, distal, diamond coated                | 33  | 0.571.6821  | 33A   | 1.006.1975  |
|                                       | SONICflex micro tip set                                                      | Set | 0.571.0421  | Set A | 1.006.2023  |
|                                       | SONICflex bevel, mesial, diamond coated                                      | 58  | 0.571.7392  | 58A   | 1.006.1988  |
|                                       | SONICflex bevel tip, distal, diamond coated                                  | 59  | 0.571.7382  | 59A   | 1.006.1990  |
| Preparation                           | SONICflex bevel set A                                                        | Set | 0.571.0391  | Set A | 1.006.2030  |
|                                       | SONICflex Carix D tip, spherical, small, diamond coated                      | 42  | 0.571.7105  | 42A   | 1.006.1980  |
|                                       | SONICflex Carix D tip, spherical, large, diamond coated                      | 43  | 0.571.7167  | 43A   | 1.006.1981  |
|                                       | SONICflex Cari EX D set A                                                    | Set | 0.571.7378  | Set A | 1.006.2025  |
|                                       | SONICflex Carix TC tip, spherical, small, hard metal                         | 71  | 1.000.7362  | 71A   | 1.006.2002  |
|                                       | SONICflex Carix TC tip, spherical, large, hard metal                         | 72  | 1.000.7363  | 72A   | 1.006.2004  |
|                                       | SONICflex Cari EX TC set A                                                   | Set | 1.000.7379  | Set A | 1.006.2032  |
|                                       | SONICflex prep gold tip, mesial, diamond coated                              | 49  | 0.571.7212  | 49A   | 1.006.1983  |
|                                       | SONICflex prep gold tip, distal, diamond coated                              | 50  | 0.571.7222  | 50A   | 1.006.1984  |
|                                       | SONICflex prep gold set A                                                    | Set | 0.571.0321  | Set A | 1.006.2028  |
| Surgery                               | SONICflex prep ceram tip, mesial, diamond coated                             | 51  | 0.571.7262  | 51A   | 1.006.1985  |
|                                       | SONICflex prep ceram tip, distal, diamond coated                             | 52  | 0.571.7272  | 52A   | 1.006.1986  |
|                                       | SONICflex ceram Set A                                                        | Set | 0.571.0341  | Set A | 1.006.2029  |
|                                       | SONICflex prep CAD/CAM tip, mesial, diamond coated                           | 34  | 1.002.1984  | 34A   | 1.006.1977  |
|                                       | SONICflex prep CAD/CAM tip, distal, diamond coated                           | 35  | 1.002.1986  | 35A   | 1.006.1979  |
|                                       | SONICflex CAD CAM set A                                                      | Set | 1.002.1988  | Set A | 1.006.2024  |
|                                       | SONICflex cem tip                                                            | 12  | 0.571.5431  | 12A   | 1.006.2021  |
|                                       | SONICflex crown prep 97 radius                                               | 97  | 1.008.6383  | 97A   | 1.008.6384  |
|                                       | SONICflex crown prep 98 level                                                | 98  | 1.008.6385  | 98A   | 1.008.6386  |
|                                       | SONICflex bone tip, spherical, large, diamond-coated, sinus floor evaluation | 81  | 1.004.3876  | 81A   | 1.006.2008  |
|                                       | SONICflex bone tip, large ball, non-diamond coated, Sinus Floor Evaluation   | 82  | 1.004.3877  | 82A   | 1.006.2010  |
|                                       | SONICflex bone tip, saw, sagittal                                            | 83  | 1.004.3878  | 83A   | 1.006.2012  |
|                                       | SONICflex bone tip, saw, axial                                               | 84  | 1.004.3879  | 84A   | 1.006.2014  |
|                                       | SONICflex bone tip 87 A, membrane disc                                       | 85  | 1.006.0645  | 85A   | 1.007.1624  |
|                                       | SONICflex bone tip, scratcher                                                | 86  | 1.006.0741  | 86A   | 1.007.1625  |
|                                       | SONICflex bone tip, saw                                                      | 87  | 1.006.1275  | 87A   | 1.007.1627  |
|                                       | SONICflex bone tip, rectangular, for bone splitting                          | 80  | 1.004.3875  | 80A   | 1.007.2006  |
|                                       | SONICflex bone tip set                                                       | Set | 1.004.3874  | Set A | 1.006.2033  |

KaVo. Dental Excellence.